



## EEG: Disconnect & Reconnect for Procedures/ Transfers

- **DO NOT** DISCONNECT THE GRAY CABLES FROM THE EEG DEVICE
  - This often results in lost data!



OR



### For Procedures or Transfers:

- **ALWAYS** UNCLIP THE ELECTRODE ARRAYS AT PATIENTS HEAD
  - Carefully unclip the **YELLOW TAB CONNECTORS FOR EACH ARRAY** LABELED: **1, 2, 3, and** remove the EKG lead taped on the patient's chest
- **REMINDERS:**
  - Reconnect labeled arrays & EKG lead **ASAP** upon return to resume monitoring



"Yellow clip"

- When relocating the EEG equipment: *Follow the same as above, do not unplug grey cable, use the electrode arrays at the patient's head. Unplug the power supply to the EEG machine and move with patient. Upon arrival to new location with equipment and patient reconnect patient to EEG via the Electrode arrays and **DON'T FORGET TO PLUG INTO A RED POWER OUTLET WHN AVAILABLE IN CASE OF POWER OUTAGE.***

