

Powered Air Purifying Respirator (PAPR)

- **PAPRs are supplied by Equipment Distribution. PAPRs can be requested by placing a Service Central Medical Equipment Request:**

- Under "Requesting Equipment", select Add
- Equipment Type: Other
- Quantity: # of PAPRs needed
- Request Priority: How Quickly Needed
- Save, then write "PAPR(s)" in the Additional Comments section



- **Please return PAPRs to Equipment Distribution when no longer needed**
 - Return via Service Central Medical Equipment Request, select "Returning Equipment"

- **Alternatively, you may call Equipment Distribution to request or return PAPRs @ 828-213-0200**

PAPR: How to

Click or
Scan



How to use

- Donning should take place in a clean, hazard-free environment
 - Place the blower unit and belt around the waist and adjust to good fit
 - Turn on the blower unit
 - Place head cover on head with padded part of head strap centered on forehead
 - Enter contaminated (infected) area breathing normally
- Remove the blower unit and head cover in a safe, hazard-free area
 - Remove the head cover (hood)
 - Remove the belt and blower from waist
 - Turn blower off

How to clean/maintain

- Wipe PAPR blower unit, battery pack and head cover with hospital approved disinfectant—e.g. purple top super sani cloth
- Do not spray or submerge this equipment
- Each hood is meant to be reused by the same person if properly cleaned and maintained
- After cleaning place hood back in the round cardboard canister or a bag and mark your name on it
- Dispose of head cover if it is badly soiled, torn, elastic is no longer tight or face shield is cracked or bent
- Never try to clean filters or blow into the filter
- Contact equipment distribution to change filters, treat used filters as contaminated